

18"

9"


9"



I'M A PAN Optimist

CATARACTSURGERY.COM

I SEIZE LIFE WITHOUT CATARACTS

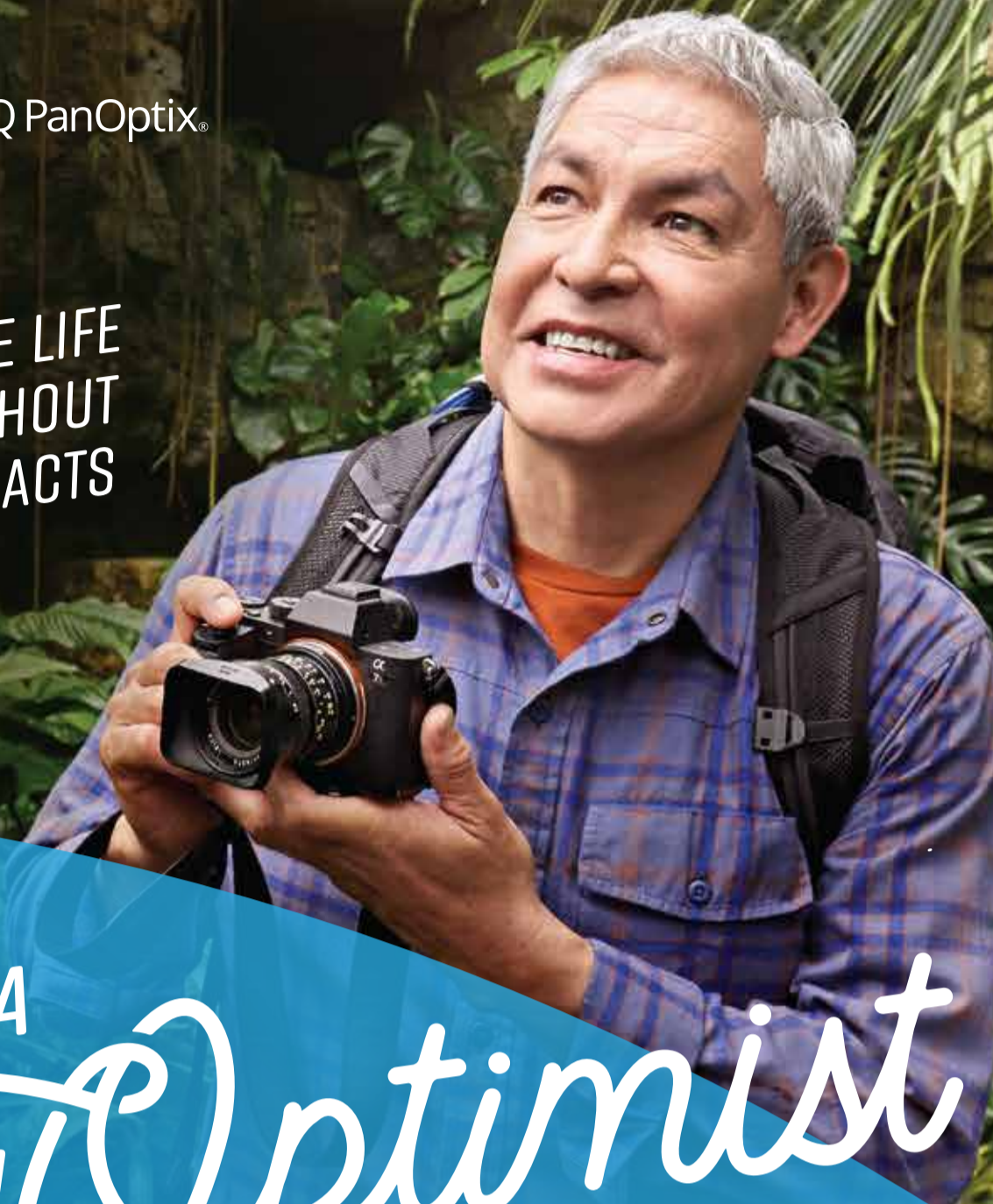

I'M A PAN Optimist



ACRYSOF® IQ PANOPTIX® FAMILY OF TRIFOCAL IOLs IMPORTANT PRODUCT INFORMATION

CAUTION: Restricted by law to sale by or on the order of a physician.

DESCRIPTION: The AcrySof® IQ PanOptix® Trifocal IOL is a type of multifocal IOL used to focus images clearly onto the back of your eye (retina) to allow clear vision after the cataract removal. In addition, the center of the AcrySof® IQ PanOptix® Trifocal IOL allows for better near (reading) vision and intermediate (computer work) vision versus what a monofocal lens would provide.

There is a chance that you still may need glasses for distance, intermediate, and/or near vision. You will get the full benefit of the AcrySof® IQ PanOptix® Trifocal IOL when it is placed in both eyes. Please discuss with your eye doctor whether this is the right IOL for you.

POTENTIAL SIDE EFFECTS: Due to the design of multifocal IOLs, there are some side effects that can be associated with the AcrySof® IQ PanOptix® Trifocal IOL models. These may be worse than with a monofocal IOL, including visual disturbances such as glare, rings around lights, starbursts (rays around light sources), and reduced contrast sensitivity (decrease in ability to distinguish objects from their background, especially in dim lighting). These side effects may make it more difficult to see while driving at night or completing tasks in low lighting conditions, such as at night or in fog or in a dimly lit room, after surgery as compared to before surgery.

Further, a toric IOL corrects astigmatism only when it is placed in the correct position in the eye. There is a possibility that the toric IOL could be placed incorrectly or could move within the eye. If the toric lens is not positioned correctly following surgery, the change in your astigmatism correction by the IOL, along with any necessary correction with glasses, may cause visual distortions. If the lens rotates in your eye, you may need additional surgery to reposition or replace the IOL.

ATTENTION: As with any surgical procedure, there are risks involved. Prior to surgery, ask your eye doctor to provide you with the Patient Information Brochure for the proposed AcrySof® IQ PanOptix® Trifocal IOL to be implanted. This document will further inform you of the risks and benefits associated with this IOL. Discuss any questions about possible risks and benefits with your eye doctor, as well as your medical condition and any eye disease you may have.

Reference: 1. AcrySof® IQ PanOptix® Directions for Use.

© 2019 Alcon Inc. 11/19 US-ACP-CEI-1900082

Alcon

Alcon

Less Glasses. More You.

Seize your once-in-a-lifetime opportunity to gain clear, complete vision with the PanOptix® Lens, the first and only trifocal cataract replacement lens in the US.

score line

die line

REDEFINE YOUR VISION

By speaking to your doctor about the PanOptix® Lens, you've taken the first step toward a once-in-a-lifetime opportunity that opens up a million more.

First and only trifocal lens in the United States

Enhanced vision for clear, complete focus

Reduced need to wear glasses!

Available with astigmatism-correcting technology



See life differently with the PanOptix® Lens.

FIND OUT MORE AT CATARACTSURGERY.COM

glue tab

glue tab

12"

5"



18"

9"

9"

Paint Scissors + Adhesives Drawin

I FIND MAGIC  
IN THE DETAILS



12"

score line

die line

glue tab

5"

glue tab