

# VERION™ Digital Marker

## Device Cleaning

**Alcon**  
SEE BRILLIANTLY

Date: June 1<sup>st</sup> 2020

The United States Centers for Disease Control and Prevention (CDC) includes 70% Isopropyl alcohol in its recommendations for disinfecting against COVID-19. The CDC has also reported on the use of bleach as an effective disinfection agent for COVID-19. A full list of approved disinfection agents can be found at [www.epa.gov](http://www.epa.gov). For your medical devices, it is important to note that only cleaning products that have been approved by the manufacturer should be used.

The below information is intended to provide guidance on cleaning and disinfection for the VERION™ Digital Marker for LenSx® and Microscope

- Unplug the main power cable from the power supply prior to cleaning.
- Alcon recommends using Isopropyl as disinfectant. Use a solution containing at least 70% Isopropyl alcohol, as recommended by the CDC.
- Clean the outer surfaces of the Digital Marker with a soft cloth drenched in 70% Isopropyl alcohol and antistatic cleaner.
- Do not use wet tissues and solvents such as acetone to clean the device.
- To prevent damaging the device console or its accessories, do not spray or pour cleaning agents directly on their housings.
- Avoid moisture getting inside of the instrument since this is likely to create electrical hazards.
- To clean displays, use commercial cleaning cloths for computers and computer screens only. Do not apply extensive pressure.
- Make sure to fully wipe off any cleaning agent residues.

The American Academy of Ophthalmology has provided guidelines for the use of personal protective equipment (PPE) to help reduce the risk to patients and employees of virus transmission. In addition to our existing device-specific directions for use and manuals of operation, Alcon is developing recommendations on the use of optional device-specific protective measures and will make these recommendations available to our customers through our information website.

Alcon cannot guarantee sterility or exclusion of any risk of infection of COVID-19 during the use of its devices.



VERION™ Digital Marker

